

# OnePower™

## READERS

**Care. Cleaning. Warranty.**

**ATTENTION:** Ready-to-wear non-prescription glasses are not intended to replace prescribed corrective lenses or examinations by an eye care professional. Continuous eye check-ups are necessary to determine your eye health status and vision needs.

### **For California Residents**



**WARNING:** This product can expose you to nickel<sup>1</sup>, which is known to the State of California to cause cancer. For more information go to [www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov).

## Congratulations on your purchase of One Power Readers!

We truly hope you will enjoy the freedom you will have when you can move from one task that needs up-close vision with precision to another.

One Power Readers take the guesswork out of reading glasses. Now, you can read small print and text messages, thread a needle, tie a fishing line, and read your computer monitor all with the same pair of One Power Readers.

One Power Readers come in either round frames or square. We have basic styles in black and tortoise and high-fashion styles and colors in blue, red, and smoky grey. If you would like to order more One Power Readers or try some of our other styles, simply go to our website at [www.OnePowerReaders.com](http://www.OnePowerReaders.com) or call our customer service number at 973-287-5122.

Congratulations again on your order of One Power Readers, and thank you so much for your purchase.

Sincerely,  
Customer Service  
One Power Readers

## Cleaning

Cleaning and storing your One Power Readers is very simple.

1. First, rinse your glasses with tap water. This will help remove any dust or dirt that has settled on them.
2. Make or choose an eyeglass cleaner solution. Most premixed lens cleaning solutions consist of alcohol and water. You can make a solution using one part alcohol and one part water, or purchase a premixed lens cleaner and use that.
3. Use a microfiber cleaning cloth. The microfiber cleaning case that comes with your One Power Readers is perfect for this step, or you can use any other microfiber cloth. A cotton cloth can also be used.
4. Spray the front and back of the lens with your cleaning solution.
5. Gently wipe the lens dry with your cloth. It is best to wipe from one side using a circular motion.
6. You should also wipe the nose pad and the hinges to remove any oil that may have gotten on them.

## Storage

Follow these tips and tricks for storing and caring for your One Power Readers. These tips can help you keep your One Power Readers looking and feeling like new year after year.

1. When you are wearing your reading glasses, it can be a good idea to attach them to a chain or a lanyard that hangs around your neck. This can help keep them handy and always within reach. It also can help you avoid misplacing or losing them.
2. Take your One Power Readers off with two hands. Put one hand on each end of the frame and pull gently. This helps prevent the frames from stretching and the screws from becoming loose.
3. Try not to wear your reading glasses on top of your head because it can distort the shape of your glasses.
4. When not wearing your glasses, it is a good idea to return them to the microfiber storage case or a solid protective glasses case.

5. When putting your One Power Readers down, always rest them on a flat surface and place them with the frames upside down.
6. Avoid leaving your One Power Readers on the dashboard or seat of your car. The direct intense heat can distort their shape.
7. Try to avoid letting perspiration or hair spray stay on the lenses for extended periods of time. If you are sweating or have just sprayed your hair, it is a good idea to wipe off your lenses.

## Manufacturer's Limited Lifetime Warranty

The manufacturer warrants that your One Power Readers is free of defects in materials and workmanship.

This warranty is valid only in accordance with the conditions set forth below:

1. This warranty applies to consumer use only, and is void when the product is used in a commercial or institutional setting.
2. The warranty extends only to the original consumer purchaser and is not transferable. In addition, proof of purchase must be demonstrated. This warranty is void if the product has been subject to accident, misuse, abuse, improper maintenance or repair, or unauthorized modification.
3. This limited warranty is the only written or express warranty given by the manufacturer. Any implied warranty of merchantability or fitness for a particular purpose on this product is limited in duration to the duration of this warranty. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.
4. Repair or replacement of the product (or, if repair or replacement is not feasible, a refund of the purchase price less any shipping, handling or processing fees that you paid) is the exclusive remedy of the consumer

under this warranty. The manufacturer shall not be liable for any incidental or consequential damages for breach of this warranty or any implied warranty on this product. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

5. This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

### **Procedure for Exercise of Warranty Rights:**

To return the product under the limited lifetime warranty, the original purchaser must package the product securely and include the following information: (1) name, (2) address, (3) the reason for return and request for replacement, and (4) either proof of purchase or order number. Place the information inside an envelope, place the envelope inside the package with the unit, include a check for \$5.99 for each set of glasses to cover processing and handling, and mail the package to the address listed below.

Return Address:

Tristar Products, Inc.

500 Returns Road

Wallingford, CT 06495

# OnePower™

## READERS



Distributed by:  
Tristar Products, Inc.  
Fairfield, NJ 07004  
©2018 Tristar Products Inc.

MADE IN CHINA

[www.onepowerreaders.com](http://www.onepowerreaders.com)  
OPR\_IB\_TP\_ENG\_V1\_180815