

2-in-1 LEG & BACK RELIEF WEDGES

INSTRUCTIONS FOR USE

1. Remove the wedge from the packaging and unfold the wedge.
2. Pull the valve out of its recessed position and remove the cap from the valve. Insert the tip of the pump into the valve (Fig. A).



3. Pinch the valve lightly with your thumb and forefinger. Use the palm of the same hand to steady the tip of the pump. Use the pump to push air into the wedge until inflated to the desired fullness.

NOTE: Inflating the wedge 80% full is ideal for most people. When 80% full, pushing the wedge with your finger should leave a moderate indentation. A completely inflated wedge is too firm for most people.

4. Remove the pump from the valve. Place the cap on the valve and push the valve into the wedge.

TO DEFLATE THE WEDGE, pull on the exposed tab to release the valve from the recessed position within the wedge. Remove the cap from the valve. Pinch the valve opening together with two fingers to release the air.

IMPORTANT SAFEGUARDS

- **NOT A TOY.** Only use as instructed.
- Do not sit, stand, or jump on the unit.
- Do not over-inflate.
- Only use the air pump that is included.



TO CLEAN, use a damp cloth to wipe away any dirt or spilled liquids.

SPOT CLEAN WATER ONLY
AIR DRY ONLY



90-Day Money-Back Guarantee

If you decide that you don't love this product, your order is covered by our 90-Day Money-Back Guarantee. You have 90 days from the day you receive your product to request a replacement product or refund of the purchase price minus processing and handling fees.

To receive a refund or replacement, you must return the product to us at your expense. To return the product, call customer service at 1-973-287-5174 and request a return merchandise authorization number. Pack the product carefully and include in the package a note with (1) your name, (2) mailing address, (3) phone number, (4) email address, (5) reason for return, and (6) proof of purchase or order number, and (7) specify on the note whether you are requesting a refund or replacement. Write the return merchandise authorization number on the outside of the package.

Send the product to the following return address:

Contour Back Relief Wedge
Tristar Products
500 Returns Road
Wallingford, CT 06495



Distributed by:
Tristar Products, Inc.
Fairfield, NJ 07004
Made in China
© 2019 Tristar Products, Inc.

CLBRW_IB_TP_ENG_V1_190917